

# 7 Simple Steps to Independence.

## Car/Boat/RV Usage

After you are familiar with operating your OxyGo at home, you are ready to take your OxyGo on the road.

As always, be a careful driver. DO NOT RUSH OR HURRY to set up your OxyGo in the car. Take your time and follow the steps outlined to the right. The OxyGo can operate either by battery or the DC Power Cord while in your car, boat, or RV.

### CAUTION

Make sure your OxyGo is secure before driving.

## Applied Home Healthcare Equipment

Applied Home Healthcare Equipment LLC  
28825 Ranney Parkway  
Westlake OH 44145 USA

Your Equipment Provider:

Your Physician:



Plug the DC Power cord into the cigarette lighter or auxillary DC power supply.



Plug the power output cord into your OxyGo and position your OxyGo securely inside the car. Operate your OxyGo as normally.



Applied Home  
Healthcare Equipment  
Westlake, OH 44145



Now that you have unpacked each box, you have everything you need to start using your OxyGo (1400-1000). The set-up of your OxyGo is quick and simple. However, before using the OxyGo, please read the entire User Manual.




1 Place your OxyGo in a well ventilated location. Ensure both particle filters are in place.



2 Insert the OxyGo battery by sliding battery into place until the latch returns to the upper position.



3 Connect the AC Power Supply cord to the OxyGo.

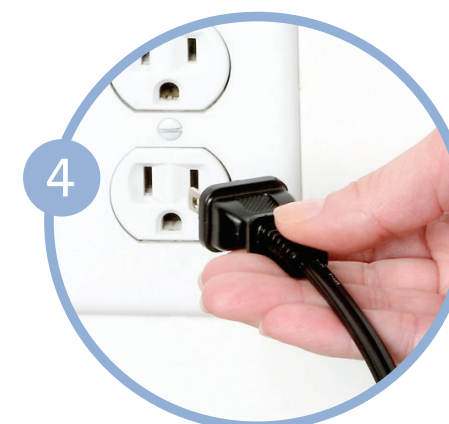

4 Plug the Power Supply AC cord into the nearest electrical outlet.



5 Connect the nasal cannula tubing into the metal nozzle on your OxyGo. Oxygen will only be delivered while you are breathing through the nasal cannula connected to the device.



6 Turn on the OxyGo by pressing the  button.



7 Set the flow setting to the flow rate prescribed by your clinical professional. The flow setting can be changed by pressing the triangular flow setting buttons that are marked with “+” and “-” symbols. Position the nasal cannula on your face and breathe normally through your nose.

**Applied Home  
Healthcare Equipment**  
Westlake, OH 44145

NOTE: You must operate the OxyGo on AC Power for approximately 3 hours to fully charge the battery.